

## Pet Dog Leg Protector Hip Auxiliary Correction Belt

### Product Features/Functions:

1. The dog's thigh and hip joint protector is made of neoprene, which feels delicate, soft and elastic
2. The leg support sheath has a shockproof effect, and it can be worn after injury or surgery to effectively prevent secondary injuries when the dog is injured
3. Relieves pain from cruciate ligament injuries, osteoarthritis and kneecap problems
4. Helps treat the loss of stability caused by arthritis.
5. Suitable for small, medium and large dogs with short legs.

### use:

1. Great for wrapping leg wounds. Just wrap the wound and wrap it with our support frame dog leg wrap to prevent secondary infection of the wound caused by dog licking or touching water.
2. Your dog may suffer secondary injuries from a knee injury, which requires a dog leg guard.
3. Provide maximum support and stability. Helps with: Stable arthritis after fatigue injury Supports any postoperative strain
4. Pay attention to remove the sheath 2-3 times a day, let the legs breathe at night

# PRODUCT INFORMATION



NAME

**Dog Hip Brace Support** (PD60176)

---

MATERIAL

**Polyester fibre**

---

COLORS

**Black**

---

SIZE

**S、 M、 L**

---



## Adjustable leg brace

Adjustable neoprene strap on legs

...

## With band



For more flexible movement

...



# Size Chart

\*Please refer to the following figure to measure the thigh circumference

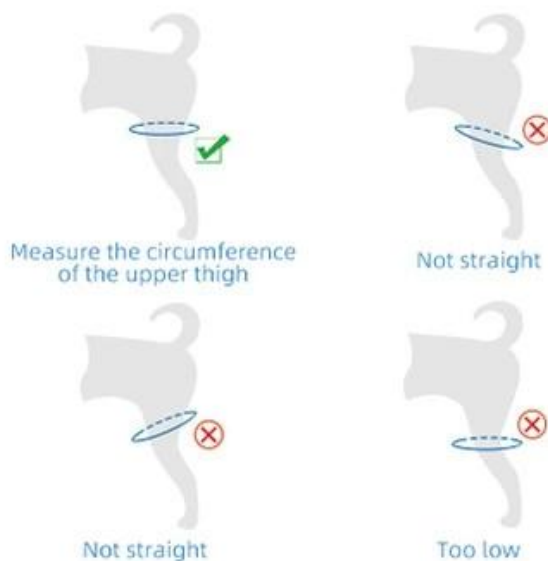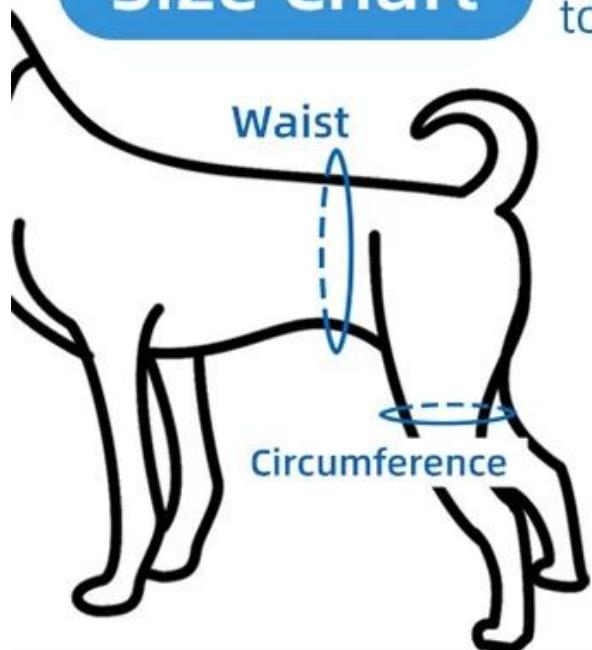

Size	Waist	Circumference
S	25-46cm/9.8"-18.1"	7-11cm/2.8-4.3"
M	38-68cm/15"-26.8"	10-15cm/3.9"-5.9"
L	58-89cm/22.8"-35"	16-25cm/6.3"-9.8"

\*If your dog is between two sizes, Please choose the smaller one.

# PRODUCT DETAILS



## Adjustable buckle

Adjustable fitting with adhesive strap



## Breathable mesh

Adjustable air mesh waist



# WHICH DOGS ARE PRONE TO ILLNESS

---



Normal Hip



Moderate  
Hip Dysplasia



Severe  
Hip Dysplasia



St. Bernard (47%)



ROTTWEILER (20%)




Golden retriever (20%)



Collie (19%)

This is also true of Pug, Bulldog and other breeds

# PREVENT HIP DYSPLASIA

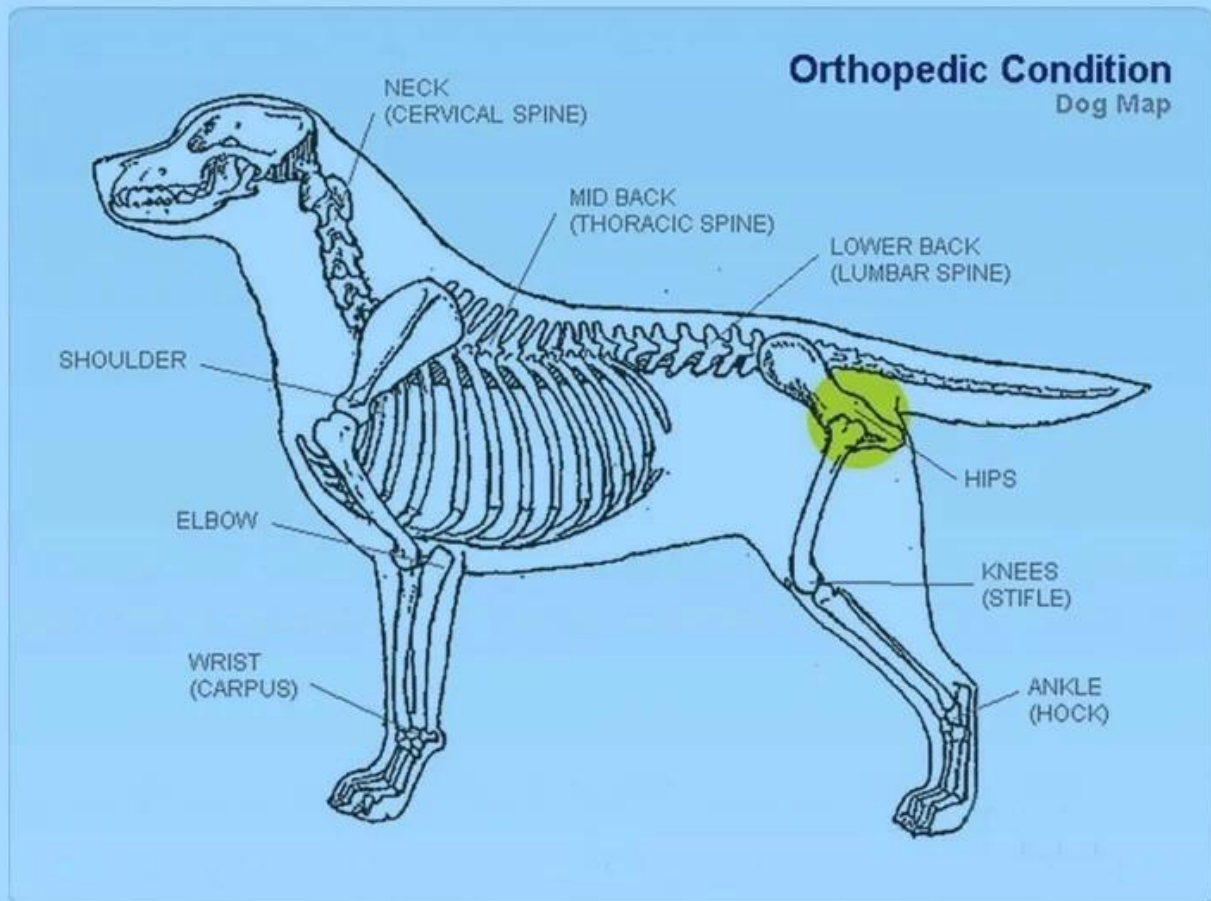

Applicable to the treatment stage of hip joint dysplasia. Adjust the range of motion, reduce the pain caused by the motion on the hip joint, and have a certain protective and corrective effect; Preventing on the postoperative recovery of dogs who need surgical treatment and correction.

## USING DOG HIP BRACE SUPPORT

---

Dog Hip joint support is designed for dogs with hip joint dysplasia or similar conditions, which has certain preventive and improving effects. With adjustable waist strap to keep the leg brace in place. Adjustable leg brace . Available in 3 sizes





# DOG HIP BRACE SUPPORT

---

Helps prevent and treat hind leg crossing in your dog.

The Cross-Assist may be adjusted to fit the width of your pets' legs and is a training tool for those pets who cross their hind legs. The corrective training tool allows your pet to move naturally while keeping the hind legs apart at the same time, retraining your dog to walk correctly.

